Japan Polymer Technology

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LapaSta has been a game-
changer in the concept of
dry boxes.

Shifting the view for new beginnings

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It takes less than 10 seconds
for setup and cleanup before
training.

You can carry it in your bag.
Frees you from complex setup.

Japan Polymer Technology

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Price ¥24,000.

2 pcs./set with port

(excluding delivery fee
and tax)

Shifting the view for new beginnings

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specifications

Product name	LapaSta
Height & Weight	210mm・175g/pc
Sucker type	Φ70mm PVC made Lever action
Metal ball head	360-degree rotation
Needle hole	Rubber made
Arm	Aluminum made
Port	Includes, The colors are also sometimes changed
Composition	2-piece kit

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3

How to Set up

Recommended location	Glossy surface to which the suction cups can be attached. We recommend the use of JPTC POM-KITCHI plate (sold separately) for mounting surfaces that do not have suction cups.
Check before use	Make sure the suction cup lever is up. Make sure the surface is not contaminated with dust, oil, chemicals, mold release agents, etc.
Get ready for use	<ol style="list-style-type: none">1. Press the top of the suction cup to the ground with your finger and lower the suction cup lever (Images 1, 2, and 3).2. Release the arm knob and position the forceps entry while holding the arm.3. To hold the arm position, tighten the knob while supporting the arm with your hand.

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6

Put Back

1. Press fingers on top of suction cup and lift suction lever with other fingers (Images 4 and 5).
2. When you place the tip of your nail between the suction cup and the surface of the table, the suction cup is lifted (Image 6).
3. Release the arm knob.
4. Keep away from direct sunlight and chemicals.

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Notes

- Not for Medical Use
- After the arm is fixed, do not force it to move.
- If forceps or a needle holder is inserted into the LapaSta, there is a risk that it will fall and be damaged.
- Do not apply mold release compound, oil, acid or alkali to the suction cups.
- Please do not touch the suction cup material as it is very sensitive to sweat.
- Rough table surfaces will not allow installation.
- Do not insert forceps or needle forceps with an open handle into the insertion port.
- Do not pull on the LapaSta which holds it.
- If the rubber with the port is damaged, it will tear.
- If used in a wet lab, make sure it is not contaminated.
- Be careful not to break the suction cups.
- After use, remove it from the table, loosen the knob and put it away.
- Be sure the table surface is clean before use.

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Regarding Our Quality Control

- Use a jig to adjust the connection between the arm and the suction cup to ensure there is no looseness.
- Visually inspect the suction cups for cuts or dirt.
- Visually inspect for cracks in the rubber.
- Visually verify rubber fits into the arm.
- Visually inspect for cracking when the Port is inserted into the rubber.
- Make sure the suction cup is working by repeatedly releasing and tightening the suction cup lever.
- Place the LapaSta on the acrylic plate. Make sure it does not fall over for 4 hours.
- Make sure the metal head ball is spinning. Loosen the arm knob.
- After LapaSta is fixed, insert the needle holder through the port and make sure that the sucker does not peel off when operated.

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Registration: No. 5701457

September 12, 2014

It is prohibited to post LapaSta® on
websites, social networking sites,
advertisements, etc.

without our permission.

Shifting the view for new beginnings

Japan Polymer Technology

Shifting the view for new beginnings

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